



Head Office
RAPP Australia Pty Ltd
Unit 1, 22 Hick Street,
PO Box 120, Lara,
Victoria, Australia 3212
ABN 35 054 189 365

Office: +61(0)3 5284 0222
Fax: +61(0)3 5282 4892
sales@rappastralia.com.au

Cleaning Guidelines for Chemical Disinfection

Cleaning Safety:

Minimum Personal protective equipment that should be worn when undertaking chemical disinfection are rubber gloves, Safety glasses, mask, coat.

Emergency Procedure:

In case of exposure to eyes or skin, flush the area for 15 to 20 minutes with running water.

Cleaning Chemical:

1% liquid bleach (Sodium Hypochlorite) is recommended to disinfect the bags inside and out. This concentration is suitable to disinfect biological waste containing blood, proteins and lipids.

Household bleach is 5-6% sodium hypochlorite, therefore a 1:5 (v/v) dilution of bleach is appropriate.

Contact time: 1 minute (surface must remain moist with solution for 1 minute)

Cleaning Instructions:

Soil removal: First remove any particulate soil from the bag using a brush or cloth and soapy water as required.

Dilute bleach to 1% as above.

Dampen cloth with solution and wipe over all affected surface's – leave on surface for at least 1minute.

Wipe off solution with a clean wet cloth.

Leave open to dry.

Warranty Disclaimer:

The warranty for NEANN bags is only valid if the recommended Cleaning Guidelines for chemical disinfection are followed.

