Description of the equipment and
instructions for use

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STATION MEDUMAT

*Portable holder STATION MEDUMAT
with bracket WM 22950*

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WEINMANN

Contents

1. Description	4
1.1 Purpose	4
1.2 Function	6
2. Safety	8
3. Fitting the wall bracket	12
4. Operation	14
5. Hygienic preparation	16
6. Operational inspection	16
6.1 Test interlock	16
6.2 Test securing stirrups	18
7. Maintenance	18
8. Disposal	18
8. Repairs	20
9. Product and Accessories	24
9.1 Standard product	24
9.2 Spare Parts	24
10. Technical Data	26
11. Warranty	28

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1.1 Purpose

STATION MEDUMAT *with bracket* WM 22950

STATION MEDUMAT *with bracket* is a portable holder for devices from the MEDUMAT family.

- holds a MEDUMAT Standard, Standard a, Basic or Basic p;
- can be transported by means of the carrying handle or the carrying strap;
- can be attached to stretchers or beds by means of the securing stirrup; a strap system prevents it from falling off during transport;
- must not be used in moving vehicles.

Wall bracket STATION MEDUMAT WM 22550

Wall bracket STATION MEDUMAT is a holder for devices from the MEDUMAT family.

- holds a MEDUMAT Standard, Standard a, Basic or Basic p;
- can be fitted to a wall or ceiling, or to the side or roof inside a vehicle.

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1.2 Function

The STATION MEDUMAT can be used to hold any device from the MEDUMAT *Standard/Basic* family and secure it against loss. The securing system makes it very easy to insert and remove the MEDUMAT.

STATION MEDUMAT *with bracket* can be hung on stretchers and beds during transport and secured with straps.

The carrying strap or carrying handle on the STATION MEDUMAT *with bracket* makes it easy to transport the appropriate MEDUMAT device to wherever it is needed.

The *Wall bracket* STATION MEDUMAT is designed for permanent installation.

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2. Safety

For your own safety and in accordance with the requirements of Directive 93/42/EEC, please observe the following points:

- Read the operating instructions carefully. They are part of the appliance and must be kept available at all times.
- Separate operating instructions are supplied with the device fitted to the STATION MEDUMAT. Please observe those operating instructions and the safety rules they contain.
- Do not use STATION MEDUMAT or MEDUMAT unless you have medical training and have received instruction in respiration techniques. Incorrect use may cause serious physical injury.
- Please comply with section "5. Hygienic preparation" on page 16 to prevent infection or bacterial contamination.
- Use the STATION MEDUMAT for the described purpose only (see "1.1 Purpose" page 4).
- Have all inspection and repair work performed only by the manufacturer (Weinmann) or expert personnel.
- Malfunctions and a lack of biocompatibility may result if third-party articles are used. Please bear in mind that in these cases any warranty entitlement and liability shall lapse where the accessories recommended in the instructions for use or original spare parts are not utilised.

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Nederlands

- While transporting patients on stretchers or hospital beds, be sure to secure STATION MEDUMAT *with bracket* to the stretcher or bed so that it cannot slip. Therefore the tube diameter must be no smaller than 10 mm and no greater than 45 mm.
- STATION MEDUMAT *with bracket* WM 22950 must not be used in moving vehicles.

Important

If a MEDUMAT is to be used in vehicles, it must be secured in a holder designed for this purpose and complying with EN 1789 (*Wall bracket* STATION MEDUMAT WM 22550).

Important

When fitting the *Wall bracket* STATION MEDUMAT be sure to observe "3. Fitting the wall bracket" on page 12.

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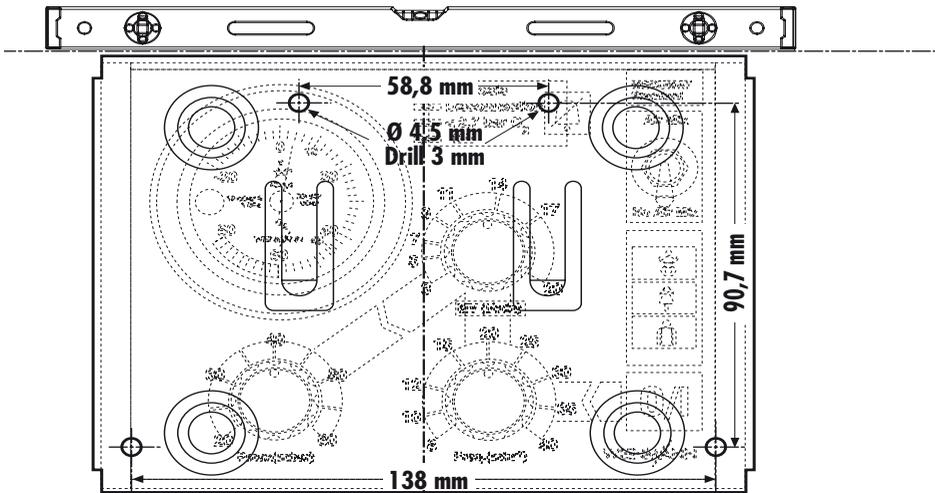
3. Fitting the wall bracket

The *Wall bracket* STATION MEDUMAT WM 22550 is screwed to a suitable point, for example to a vehicle side or roof panel.

1. Find an attachment site that is flat.
2. Mark all 4 drill holes at the attachment site.
3. Drill the holes with a 3 mm drill.
4. Attach the *Wall bracket* STATION MEDUMAT firmly with the self-tapping screws supplied.

Important

The surface to which the bracket is attached must be sufficiently strong to hold the wall bracket and the MEDUMAT securely.

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



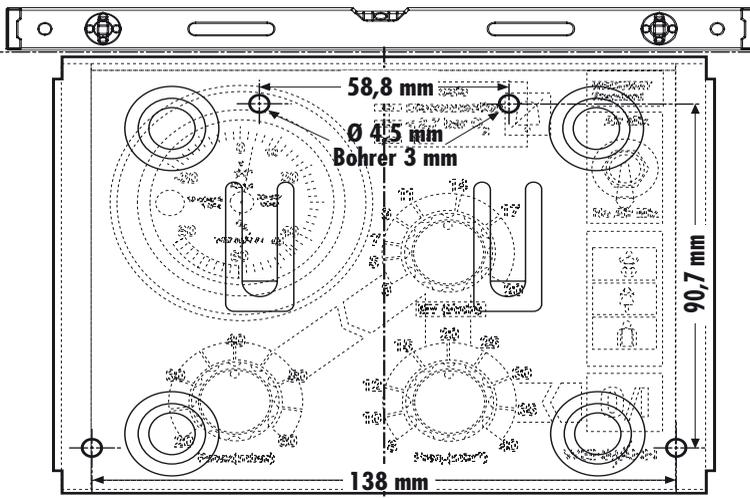
Nederlands

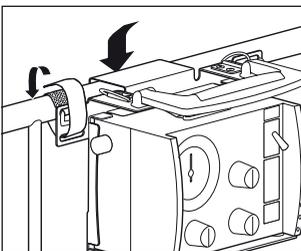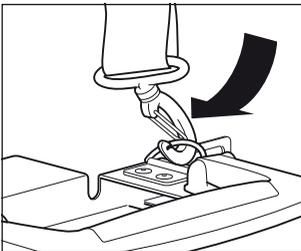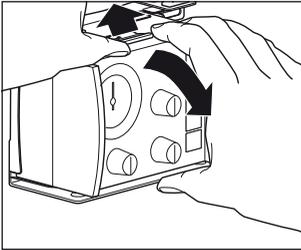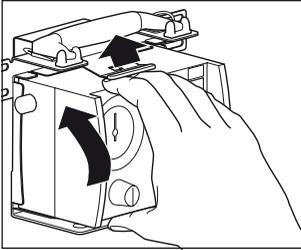4. Operation

1. To insert the MEDUMAT, place the bottom in position first, so that the lower lugs of the MEDUMAT engage in the recess in the STATION MEDUMAT. Then press the top of the MEDUMAT into the STATION MEDUMAT until it clicks firmly into place.
2. To remove the MEDUMAT, grasp it at top and bottom. Use the thumb of your other hand to push the spring tab slightly upwards, and at the same time pull first the top of the MEDUMAT and then the bottom out of the STATION MEDUMAT.
3. Use the two karabiner hooks to hang the shoulder strap in the rings on the top of the STATION MEDUMAT *with bracket*.
4. Hang the STATION MEDUMAT *with bracket* with the bracket on a bed or stretcher with a tube diameter of 10 mm – 45 mm.
To prevent the STATION MEDUMAT *with bracket* from falling off while the stretcher is being transported, secure the STATION MEDUMAT *with bracket* with the two securing straps. To do so, wind the securing strap tightly round the bar until you can attach it to itself with the velcro-type fastener after the first turn.

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Nederlands

5. Hygienic preparation

Keep the STATION MEDUMAT clean by simply wiping it with disinfectant.

Observe the instructions for the disinfectant used.

We recommend Terralin.

You are recommended to wear suitable gloves (e.g. household or disposable gloves) during disinfection procedures.

6. Operational inspection

Wall bracket STATION MEDUMAT WM 22550 and STATION MEDUMAT *with bracket* WM 22950 must be subjected to an operational inspection before each use, but at least every 6 months.

6.1 Test interlock

1. Insert a MEDUMAT *Standard/Basic* in STATION MEDUMAT and allow the interlock to engage.
2. Check that the MEDUMAT is securely supported in the STATION MEDUMAT, without opening the interlock.

MEDUMAT and STATION MEDUMAT must now be rigidly connected and must not be allowed to be separated.

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Nederlands

6.2 Test securing stirrups

(STATION MEDUMAT *with bracket* WM 22950 *only*)

1. Hang STATION MEDUMAT *with bracket* on a bed or stretcher.
2. Secure STATION MEDUMAT with the securing stirrups as described in "6. Operational inspection" on page 16

The adhesive and soft side of the securing stirrups must be rigidly connected. They must not be detached independently.

7. Maintenance

STATION MEDUMAT is maintenance-free. However, make sure that you carry out regular operational inspections (see "6. Operational inspection" page 16).

8. Disposal

Do not dispose of the unit with domestic waste. For proper disposal of the device and its components, please contact a certified waste disposal site for electronic goods. Ask your Environmental Officer or local council for the address. The device packaging (cardboard and inserts) can be disposed of in paper recycling facilities.

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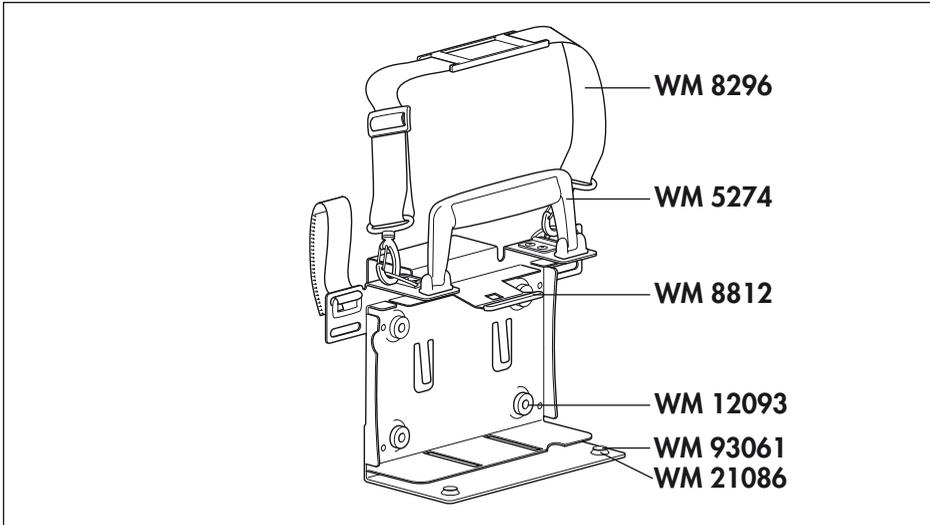
Nederlands

8. Repairs

You can perform the following operations on the STATION MEDUMAT *with bracket* and on the *Wall bracket* STATION MEDUMAT:

Replacing carrying handle, WM 5274:

1. Undo the two securing screws.
Note that there are two washers per screw.
2. Remove the old carrying handle and position the new one in the appropriate drill holes.
3. Secure the new carrying handle with the screws supplied.

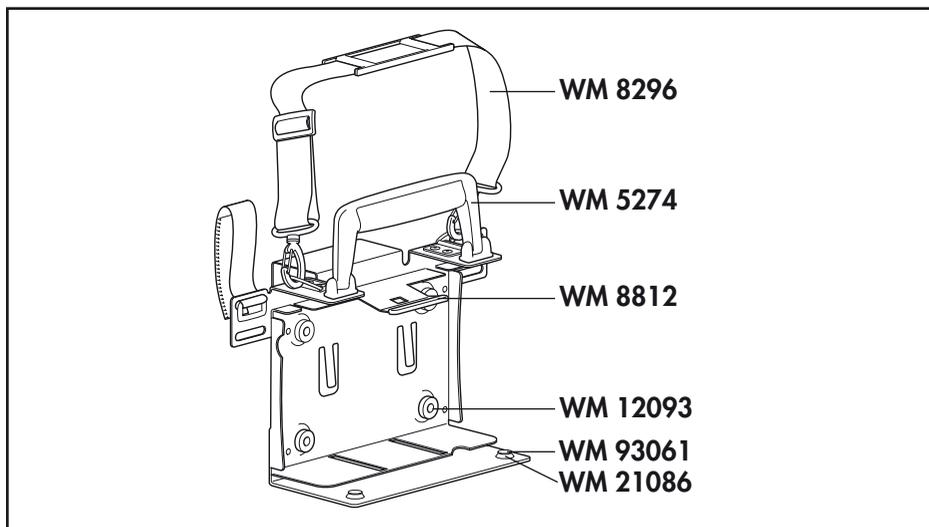
Replacing plastic cap, WM 8812:

1. Pull the loose plastic cap off the spring plate.
2. Apply a little adhesive to the inside of the new plastic cap. We recommend Loctite 601.

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3. Press the new plastic cap onto the the spring plate.

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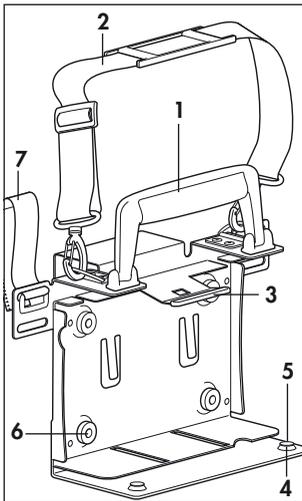
9. Product and Accessories

9.1 Standard product

1. STATION MEDUMAT *with bracket*, complete WM 22950
comprising:
 - STATION MEDUMAT *with bracket* with carrying handle and securing strap WM 22950
 - Shoulder strap WM 8296
 - Operating instructions WM 16778
2. *Wall bracket* STATION MEDUMAT complete WM 22550
comprising:
 - *Wall bracket* STATION MEDUMAT
 - Operating instructions WM 16778

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Nederlands

10. Technical Data

	STATION MEDUMAT with bracket WM 22950
Product category according to 93/42/EEC	I
Standards complied with	EN 1789
Dimensions WxHxD in mm	231x160x155
Permissible diameter for suspension	10 mm – 45 mm
Weight	0.69 kg empty, or 2 kg with MEDUMAT
Temperature range	
– Operation	–18 °C to +60 °C
– Storage	–40 °C to +70 °C

	Wall bracket STATION MEDUMAT WM 22550
Product category according to 93/42/EEC	I
Standards complied with	EN 1789
Dimensions WxHxD in mm	156x113x44
Weight	0.24 kg empty, or 1.4 kg with MEDUMAT
Temperature range	
– Operation	–18 °C to +60 °C
– Storage	–40 °C to +70 °C

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Nederlands

11. Warranty

- Weinmann warranty that the product, when used in accordance with requirements, will remain free from defects for a period of two years from date of purchase. For products whose durability is clearly indicated as less than two years, the warranty expires on the expiration date indicated on the packaging or in the user's manual.
- Claims against the warranty can be made only when accompanied by the sales receipt, which must show salesperson and date of purchase.
- We offer no warranty in the case of:
 - Disregard of usage instructions
 - Operating errors
 - Improper use or handling
 - Third-party intervention by non-authorized persons for the purpose of device repair
 - Acts of God, e.g., lightning strikes, etc.
 - Transport damage as a result of improper packaging of returned items
 - Lack of maintenance
 - Operational and normal wear and tear, which includes, for example, the following components:
 - Filter
 - Batteries and rechargeable batteries
 - Articles for one-time usage, etc.
 - Failure to use original spare parts.

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Nederlands

- Weinmann is not liable for consequential harm caused by a defect if it is not based on intention or gross negligence. Weinmann is also not liable for minor physical injury to life or limb resulting from negligence.
- Weinmann reserves the right to decide whether to eliminate defects, to deliver a defect-free item or to reduce the purchase price by a reasonable amount.
- If Weinmann rejects a claim against the warranty, it assumes no expense for transport between customer and manufacturer.
- Implied warranty claims remain unaffected by these changes.

EEC Letter of Conformity on Medical Products

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declare in sole
responsibility the

Product Name: **Carrying system for emergency
respiration unit**

Type/Model: **STATION MEDUMAT with
bracket**

being in conformity
with the respective
regulations of the
following guideline:

Directive 93/42/EEC on Medical Products

Classification:

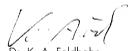
I

Standards used in particular:

DIN EN 1789: 1999

Hamburg, 03.04.2002

Dr. J. Griebahn
Executive President
Chief Executive Officer

Dr. K.-A. Feldhahn
Executive Vice President
Chief Technology Officer

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Français

Nederlands

For decades Weinmann has been developing, producing and marketing medical devices for markets around the world. In cooperation with our partners we design economic health systems for diagnosis and therapy in Sleep Medicine, Home Mechanical Ventilation, Oxygen Medicine and Emergency Medicine.

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Geräte für Medizin GmbH+Co.