

AG Cuffill manometer Quick Reference Guide

- **Before use,** make sure the device luer is clear and open to the ambient pressure.
- Turn ON by Pressing the yellow button on the right side.
- The display will show the number of readings left and then will turn to "00" (pressure)

* In case display shows value other than 00, press the yellow button for 5 sec to calibrate device is ready for use only when pressure is "0"

- **For cuff Pressure measurement** – Connect the device to the Airway cuff line as shown (Plunger all the way in),actual cuff pressure will display on screen.



- **For cuff pressure adjustment**- Connect the device to the Airway cuff line as shown (plunger between 5-10cc) and adjust cuff pressure by pushing the plunger, actual cuff pressure will appear on screen.



Display turns OFF after 60 sec